

150 mm

INFORMATION LEAFLET
(PERSONAL LUBRICANT)

Manufactured by:
PLEASURE LATEX PRODUCTS SDN. BHD.
Lot 5322, 15th Miles, Jalan Padang Gajah,
45800 Jeram, Selangor Darul Ehsan, Malaysia
Tel: 603-3250 0377 ; Fax: 603-3250 3069
Website: www.plp.com.my



Obelis s.a.
Boulevard Général Wahis 53,
1030 Brussels, BELGIUM.
Tel : +(32) 2.732.59.54
E-mail : mail@obelis.net

<u>Symbol</u>	<u>Definition</u>
	Use by date
	Date of manufacture
	Keep dry
	Keep away from sunlight
	Do not re-use
LOT	Lot or Batch Code
	Do not use if package is damaged
MD	This is a medical device
UDI	Unique Device Identification
	Importer Information
	Manufacturer Information
	Consult Instructions For Use
EC REP	Authorised Representative in the European Community/ European Union
	Distributor information
REF	Reference number

Notice to user: To report any serious incident to manufacturer /
Authorised Representative and Competent Authority of your country.

PERSONAL LUBRICANT GEL

Date of issue: 20-Mar-2025 (revision 01) – MDR 2017/745

Personal Lubricant Gel is a Light, Silky lube giving you an intensely Sensual experience! It contains a specially formulated blend of ingredients which will enhance touch between you and your partner.

This Personal Lubricant Gel is a water-based formulation and compatible with rubber condoms. Ingredients used do not have mutagenic or teratogenic effect on human.

INTENDED USERS

For layperson use.

INTENDED USE

Intended for penile and / or vaginal application. To reduce discomfort caused by vaginal dryness, and ameliorate vaginal dryness during sexual activities.

INDICATION FOR USE

To apply over the condom before sexual contact between the partners.

CONTRAINDICATION

None

WARNINGS

- If irritation or discomfort occurs, discontinue use and consult a doctor.
- This product is not a contraceptive and does not protect against pregnancy and sexually transmitted infections and HIV. To protect against pregnancy and sexually transmitted infections, this lubricant must be used with a condom.
- This lubricant shall not be used with other unintended products.
- Keep out of reach of children and avoid contact with eyes.
- This product has not been tested on pregnant women and breastfeeding mothers.

PRECAUTION

- Do not use if packaging is damaged.
- Do not swallow the lubricant..

STORAGE INSTRUCTION

- Store in a dry place.
- Keep away from direct sunlight.

INSTRUCTION FOR USE

For sachet packaging:

- Open the Personal Lubricant package by tearing gently and carefully at the serrated area.
- Apply desired amount of personal lubricant gel to intimate area or on the condom before sex.
- Reapply as needed.

For tube / bottle packaging:

- Squeeze out desired amount of personal lubricant gel and apply to intimate area or on the condom before sex.
- Reapply as needed.

DISPOSAL

- Please dispose of used Personal Lubricant package hygienically and thoughtfully. Do not flush into toilet.