

INFORMATION LEAFLET

Manufactured by:
PLEASURE LATEX PRODUCTS SDN. BHD.
Lot 5322, 15th Miles, Jalan Padang Gajah,
45800 Jeram, Selangor Darul Ehsan, Malaysia
Tel: 603-3250 0377 ; Fax: 603-3250 3069
Website: www.plp.com.my

CE
0482

EN ISO
4074 : 2015

EC REP

Obelis s.a.
Boulevard Général Wabis 53,
1030 Brussels, BELGIUM.
Tel : +(32) 2.732.59.54
E-mail : mail@obelis.net

Symbol

Definition



Use by date



Date of manufacture



Keep Dry



Keep away from sunlight



Do not re-use



Lot or Batch Code



Contains natural rubber latex



Do not use if package is damaged



Consult Instructions For Use



Manufacturer



This is a medical device



Unique Device Identifier



Importer Information



Authorised Representative in the
European Community/ European Union



Distributor information



Reference number

**Notice to user: To report any serious incident to manufacturer /
Authorised Representative and Competent Authority of your country.**

150

58

INSTRUCTION FOR USE OF MALE LATEX CONDOM

Date of issue: 10-Mar-2025 (revision 02) – MDR 2017/745

INTENDED USERS

For layperson use.

INTENDED USE

1) For Contraceptive Use:

When used in accordance with manufacturer's Instruction, condoms are effective form of contraception. However, no method of contraception can provide 100% protection.

2) For Prevention of Sexually Transmitted Diseases:

If used properly, latex condoms will help to reduce the risk of transmission of HIV infection (AIDS) and many other sexually transmitted diseases, including chlamydia infection, genital herpes, genital warts, gonorrhea, hepatitis B, and syphilis.

Do not reuse as there might be an increased risk of failure or infection.

PRECAUTION:

- Use a new condom every time you have sexual intercourse or other acts between partners that involve contact with the penis.
- If the individual container is obviously damaged, discard the condom and use a new one from an undamaged package.
- Move the condom inside the package such that it is away from the area where the package is to be torn. Tear the package and remove the condom carefully to avoid being damage by fingernails, jewellery, etc.
- If the rubber material is sticky or brittle or obviously damaged, do not use it.
- Do not use oil-based lubricants, such as those made with petroleum jelly, mineral oil, vegetable oil or cold cream, as these may damage the condom. Use water-based lubricant. Consult the pharmacist if it doubt.
- If you and your partner are using any medication in the genital area, consult your doctor or pharmacist that it is safe to use with the condoms.
- Seek medical assistance as soon as possible, at least within 72 hours should the condom leak or burst during use.
- There is a risk of choking when using a condom orally. Do not swallow the condom.
- Keep away from children.
- Choose the width of condom which is appropriate to you.
- Do not store the condom in a wallet or directly in a cloth pocket as friction can damage it.
- Choose the right condom type which is appropriate to you.
 - For oral sex, Super Strong and Super Thick Condom are recommended.
 - For homosexual practice, Super Strong Condom is recommended.
- Failure to follow information above might cause condom breakage and therefore exposes you and your partner to serious risks.

INSTRUCTION FOR USE

- Put the condom on after the penis is fully erect and before any contact occurs between the penis and the partner's body to assist in the prevention of sexually transmitted infections and pregnancy.
- Place the rolled condom on the head of the penis and leave an empty space at the end of the condom to collect the semen. Remove the air out of the tip of the condom by gently pressing the air out with one hand.
- Unroll the condom all the way to the base of the penis with the other hand. Smooth out any air bubbles.
- Stop and check if the condom is slipping off or tighten excessively onto the penis because this may lead to breakage.
- Soon after ejaculation, withdraw the penis while holding the condom firmly in place at the base of the penis to prevent the condom from slipping off.

DISPOSAL

- Use the condom once only. Wrap the used condom in tissue and dispose into a waste container.

CAUTION: THIS CONDOM IS MADE OF NATURAL RUBBER LATEX, WHICH MIGHT CAUSE ALLERGIC REACTIONS INCLUDING ANAPHYLACTIC SHOCK.

