

INFORMATION LEAFLET

Manufactured by:
PLEASURE LATEX PRODUCTS SDN. BHD.
Lot 5322, 15th Miles, Jalan Padang Gajah,
45800 Jeram, Selangor Darul Ehsan, Malaysia
Tel: 603-3250 0377 ; Fax: 603-3250 3069
Website: www.plp.com.my

CE
0482

EC **REP**

Obelis s.a.
Boulevard Général Wahis 53,
1030 Brussels, BELGIUM.
Tel : +(32) 2.732.59.54
E-mail : mail@obelis.net

Symbol



LOT



MD

UDI



EC **REP**



REF

Definition

Use by date

Date of manufacture

Keep Dry

Keep away from sunlight

Do not re-use

Lot or Batch Code

Contains natural rubber latex

Do not use if package is damaged

Consult Instructions For Use

Manufacturer

Non-sterile

This is a medical device

Unique Device Identifier

Importer Information

Authorised Representative in the
European Community/ European Union

Distributor information

Reference number

**Notice to user: To report any serious incident to manufacturer /
Authorised Representative and Competent Authority of your country.**

**INSTRUCTION FOR USE OF ULTRASONIC TRANSDUCER
PROBE COVER**

Date of issue: 26-Nov-2025(rev.01)-MDR 2017/745

READ INSTRUCTIONS BEFORE USE

INTENDED USERS

- For healthcare professional use only.

INTENDED USE

- As a protective cover in avoiding contamination of ultrasonic transducer probes by bodily fluids. To facilitate cleaning and sanitization of the probes.

Note: The Ultrasonic Transducer Probe must be cleaned and disinfected after each use, irrespective of using a Probe Cover

- Do not reuse as there might be an increased risk of infection.

INDICATION FOR USE

- These ultrasonic transducer probe covers (commonly known as probe cover) are regulated as an accessory to the ultrasonic transducer probes of the Ultrasonic Imaging System, therefore it is necessary to read the probe reprocessing instructions and/or cross-referencing of probe manuals to ensure correct usage of the probe covers use with the probes.
- If the rubber material is sticky or brittle or obviously damaged, do not use it.

DIRECTIONS FOR USE

- Check that the expiry date is current.
- Open the probe cover package by tearing gently and carefully.
- Make sure the rolled part of the probe cover to be unrolled is on the outside.
- Place the probe cover on the head of the ultrasonic transducer probe and unroll it gently all the way to the base.
- If lubrication is required, do not use oil based lubricants with this natural latex probe cover as oil-based lubricants may damage the probe cover. Use water-based lubricants.
- For single use only.

STORAGE

- Store in a cool, dry place away from direct sunlight.

DISPOSAL

- Please dispose of used probe cover hygienically and thoughtfully. Do not flush into the toilet.

**CAUTION: THIS PRODUCT CONTAINS NATURAL
RUBBER LATEX WHICH MIGHT CAUSE
ALLERGIC REACTIONS INCLUDING
ANAPHYLACTIC SHOCK.**