

INFORMATION LEAFLET
(PERSONAL LUBRICANT)

Manufactured by:
PLEASURE LATEX PRODUCTS SDN. BHD.
Lot 5322, 15th Miles, Jalan Padang Gajah,
45800 Jeram, Selangor Darul Ehsan, Malaysia
Tel: 603-3250 0377 ; Fax: 603-3250 3069
Website: www.plp.com.my

CE
0482

EC	REP
-----------	------------

Obelis s.a.
Boulevard Général Wahis 53,
1030 Brussels, BELGIUM.
Tel : +(32) 2.732.59.54
E-mail : mail@obelis.net

Symbol

Definition



Use by date



Date of manufacture



Keep dry



Keep away from sunlight



Do not re-use



Lot or Batch Code



Do not use if package is damaged



This is a medical device



Unique Device Identification



Importer Information




Manufacturer Information



Consult Instructions For Use



Authorised Representative in the
European Community/ European Union

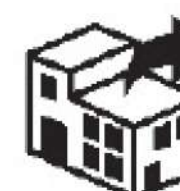

Distributor information



Reference number

Notice to user: To report any serious incident to manufacturer /
Authorised Representative and Competent Authority of your country.



PERSONAL LUBRICANT GEL

Date of issue: 01-Dec-2022(revision 00) - MDR 2017/745

Personal Lubricant Gel is a light, silky lube giving you an intensely sensual experience! It contains a specially formulated blend of ingredients which will enhance touch between you and your partner.

This Personal Lubricant Gel is a water-based formulation and compatible with condoms.

INTENDED USERS

For layperson use.

INTENDED USE:

Intended for penile and / or vaginal application. To reduce discomfort caused by vaginal dryness, and ameliorate vaginal dryness during sexual activities.

INDICATION FOR USE

To put over the condom before contact between the partners.

WARNINGS

- If irritation or discomfort occurs, discontinue use and consult your doctor.
- This product does not contain spermicide.
- This product is not a contraceptive and does not protect against pregnancy and sexually transmitted infections and HIV. To protect against pregnancy and sexually transmitted infections, the lubricant must be used with a condom.
- Keep out of reach of children and avoid contact with eyes.

PRECAUTION

- Do not use if packaging is damaged.

STORAGE INSTRUCTION

- Store in a Cool Dry Place.
- Keep Away From Direct Sunlight.

INSTRUCTION FOR USE

- Open the Personal Lubricant package by tearing gently and carefully at the pre-mark area.
- Apply desired amount of personal lubricant gel to intimate area or on the condom before sex.
- Reapply as needed.

DISPOSAL

- Please dispose of used Personal Lubricant package hygienically and thoughtfully. Do not flush into toilet.