

INFORMATION LEAFLET

Manufactured by:  
**PLEASURE LATEX PRODUCTS SDN. BHD.**  
Lot 5322, 15<sup>th</sup> Miles, Jalan Padang Gajah,  
45800 Jeram, Selangor Darul Ehsan, Malaysia  
Tel: 603-3250 0377 ; Fax: 603-3250 3069  
Website: [www.plp.com.my](http://www.plp.com.my)



**CE**  
**0482**

**Obelis s.a.**  
Boulevard Général Wahnis 53,  
1030 Brussels, BELGIUM.  
Tel : +(32) 2.732.59.54  
E-mail : [mail@obelis.net](mailto:mail@obelis.net)

Symbol

Definition



Use by date



Date of manufacture



Keep Dry



Keep away from sunlight



Do not re-use



Lot or Batch Code



Contains natural rubber latex



Do not use if package is damaged



Consult Instructions For Use



Manufacturer



Non-sterile



This is a medical device



Unique Device Identifier



Importer Information



Authorised Representative in the  
European Community/ European Union




Distributor information



Reference number

Notice to user: To report any serious incident to manufacturer /  
Authorised Representative and Competent Authority of your country.

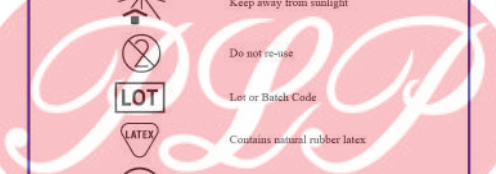

**INSTRUCTION FOR USE OF ULTRASONIC TRANSDUCER  
PROBE COVER**

Date of issue: 01-Sep-2022(rev.00) – MDR 2017/745

READ INSTRUCTION BEFORE USE

INTENDED USERS

- For healthcare professional use only.

INTENDED USE

- As a protective cover in avoiding contamination of ultrasonic transducer probes by bodily fluids. To facilitate cleaning and sanitization of the probes.  
Do not reuse as there might be an increased risk of infection.

INDICATION FOR USE

- These ultrasonic transducer probe covers (commonly known as probe cover) are regulated as an accessory to the ultrasonic transducer probes of the Ultrasonic Imaging System, therefore it is necessary to read the probe reprocessing instructions and/or cross-referencing of probe manuals to ensure correct usage of the probe covers use with the probes.
- If the rubber material is sticky or brittle or obviously damaged, do not use it.

DIRECTIONS FOR USE

- Check that the expiry date is current.
- Open the probe cover package by tearing gently and carefully.
- Make sure the rolled part of the probe cover to be unrolled is on the outside.
- Place the probe cover on the head of the ultrasonic transducer probe and unroll it gently all the way to the base.
- If lubrication is required, do not use oil based lubricants with this natural latex probe cover as oil-based lubricants may damage the probe cover. Use water-based lubricants.
- For single use only.

STORAGE

- Store in a cool, dry place away from direct sunlight.

DISPOSAL

- Please dispose of used probe cover hygienically and thoughtfully.  
Do not flush into the toilet.

**CAUTION: THIS PRODUCT CONTAINS NATURAL  
RUBBER LATEX WHICH MIGHT CAUSE  
ALLERGIC REACTIONS INCLUDING  
ANAPHYLACTIC SHOCK.**