

INFORMATION LEAFLET

Manufactured by:
PLEASURE LATEX PRODUCTS SDN. BHD.
Lot 5322, 15th Miles, Jalan Padang Terjoh,
45800 Jeram, Selangor Darul Ehsan, Malaysia
Tel: 603-3250 0377 ; Fax: 603-3250 3099
Website: www.plp.com.my

Symbol

Definition



Use by date



Date of manufacture



Keep Dry



Keep away from sunlight



Do not re-use



Lot or Batch Code



Contains natural rubber latex



Do not use if package is damaged



Consult Instructions For Use



Manufacturer

INSTRUCTIONS FOR USE

Male Latex Condom

Date of issue: 30-Jun-2022 (revision 04)

When used correctly every time you have sex, condoms greatly reduce the risk of unintended pregnancy, HIV/AIDS and some other sexually transmitted infections. Use a new condom every time you have sex and follow the instructions carefully.

Do not reuse as there might be an increased risk of failure or infection.

INTENDED USER

For In-person use.

PRECAUTION

- Choose the width of condom which is appropriate to you.
- Use a new condom every time you have sexual intercourse or other acts between partners that involve contact with the penis.
- Non-opaque individual container should not be stored outside the opaque sales packaging.
- Store condom at the proper place. Do not store the condom in a wallet or directly in a clothing pocket, as friction can damage it.
- If the individual container is obviously damaged, discard the condom and use a new one from an undamaged package.
- If the condom is sticky, brittle or obviously damaged, do not use it.
- Move the condom inside the package such that it is away from the area where the package is to be torn. Tear the package and remove the condom carefully to avoid being damaged by fingernails, jewellery, etc.
- Do not use oil-based lubricants, such as those made with petroleum jelly, baby oil, body lotion, massage oil, etc., as these may damage the condom. If additional lubrication is desired, use water-based lubricant. Consult the pharmacist if in doubt.
- If you and your partner are using any medication in the genital area, consult your doctor or pharmacist that it is safe to use with the condoms.
- Seek medical assistance as soon as possible, at least within five days, should the condom leak or burst during use.

INSTRUCTION FOR USE

- Put the condom on after the penis is fully erect and before any contact occurs between the penis and the partner's body to assist in the prevention of sexually transmitted infections and pregnancy.
- Place the rolled condom on the head of the penis and leave an empty space at the end of the condom to collect the semen. Remove the air out of the tip of the condom by gently pressing the air out with one hand.
- Unroll the condom all the way to the base of the penis with the other hand. Smooth out any air bubbles.
- Stop and check if the condom is slipping off or tighten excessively onto the penis because this may lead to breakage.
- Soon after ejaculation, withdraw the penis while holding the condom firmly in place at the base of the penis to prevent the condom from slipping off.

DISPOSAL

Use the condom once only. Wrap the used condom in tissue and dispose into a waste container.

CAUTION: This condom is made of natural rubber latex which might cause allergic reactions including anaphylactic shock.



Manufactured by:

PLEASURE LATEX PRODUCTS SDN. BHD. [199601029785/402137-W]

Lot 5322, 15th Miles, Jalan Padang Gajah,
45800 Jeram, Selangor Darul Ehsan, Malaysia.

Tel: 603-3250 0377 | Fax: 603-3250 3069

Website: www.plp.com.my

Comply with Annex 10 WHO / UNFPA Technical Specification for Male Latex Condom of WHO Expert Committee on Specification for Pharmaceutical Preparations / Comply with EN ISO 4074:2015